

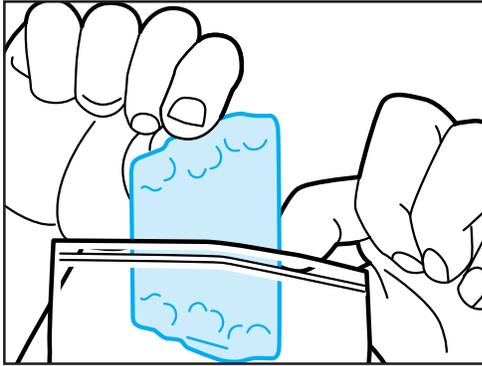


INSTRUCTIONS FOR DEVICE INSERTION

1. Remove the device from the small bag that it came in. Do not throw away the original bag as it will be used to store the used devices.

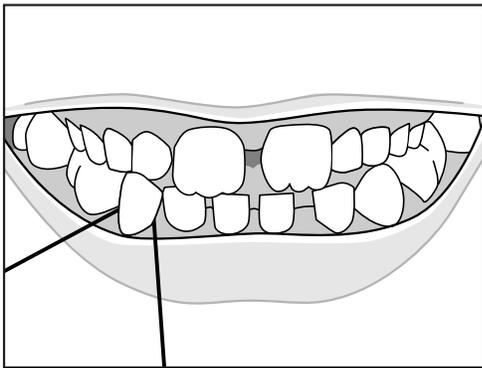


2. Ensure to brush and floss teeth before the device insertion.

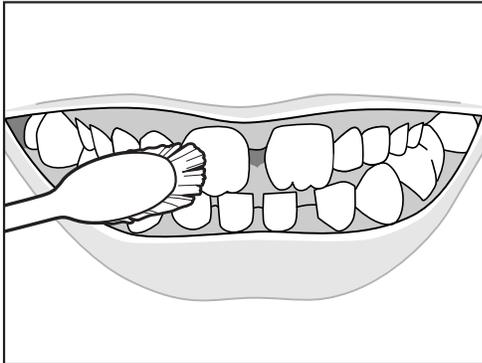


3. Confirm stage number by referencing “Understanding your new expanders and holders” section.

4. Before the device insertion:


- Clean the device with a soft bristle toothbrush, water, and a small amount of toothpaste.

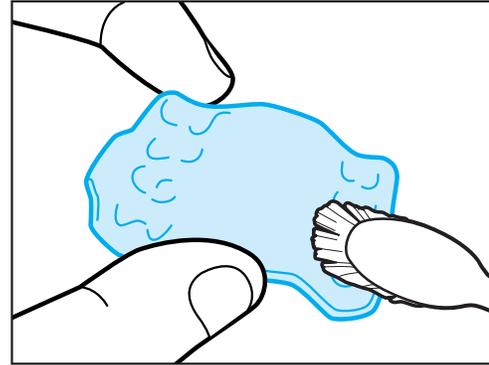


- Rinse the device thoroughly with running water.

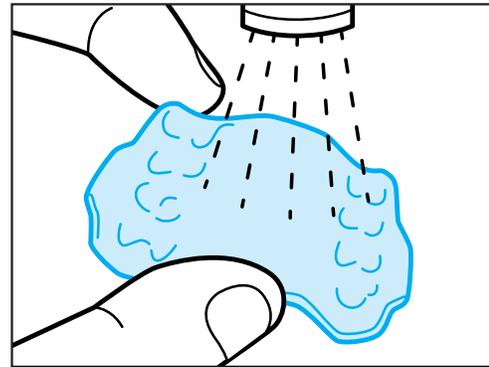


5. Place the device in the mouth under the upper back teeth, directly above the attachment wells.

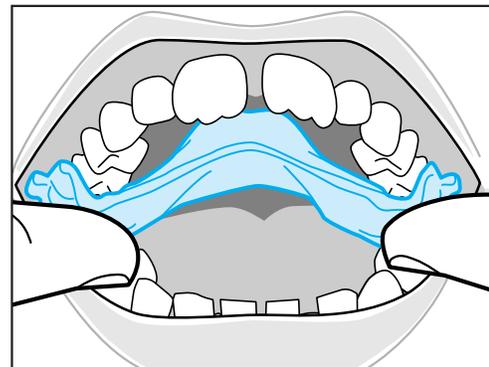


6. Push the device upward with a thumb or index finger until one side of the device is fully engaged.

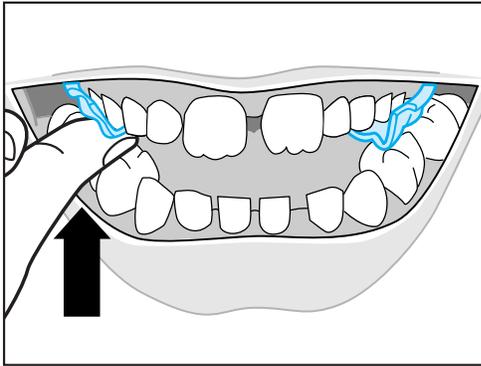

pressing up on the buccal surface of the device with a finger.

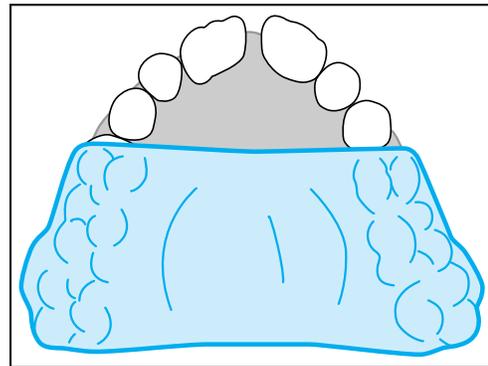


- The attachment wells of the device must be seated on the

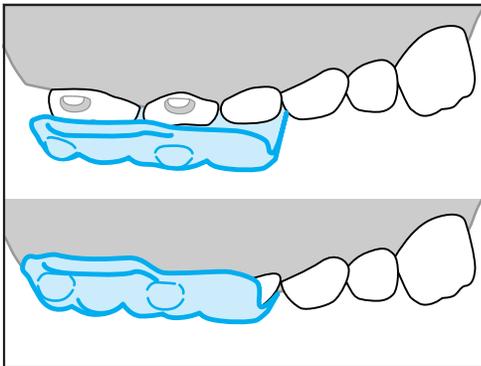

attachments.

7. Once the device is engaged on one side, remove the thumb

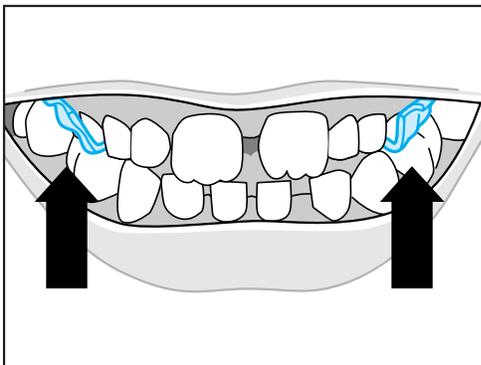


or index finger used to push the device upward.


- Bite down to engage the device on the opposite side.
- If biting down alone does not fully engage the device, then try biting down on an aligner seater or biting down while

8. The device should now be fully seated. Inspect for proper fit of the device.


INSTRUCTIONS FOR DEVICE REMOVAL

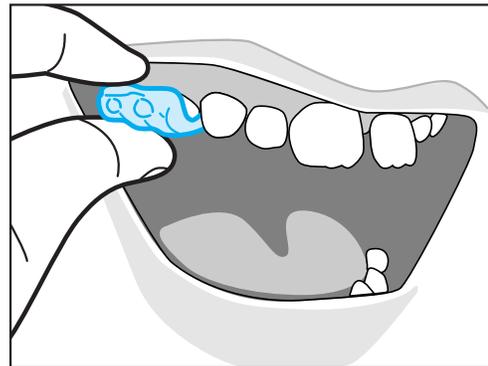

1. On one side of the device, place finger or thumb on top

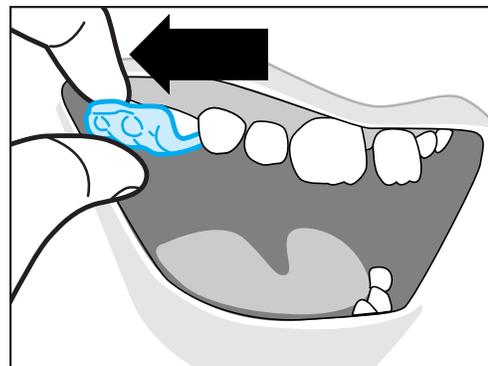


of removal handle. Another finger or thumb may be placed

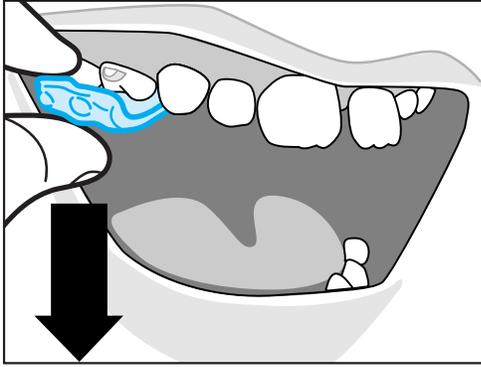


on the bottom surface of expander for better control.


2. Pull out toward the cheek to disengage from the attachment.
3. Pull down the device to remove it.

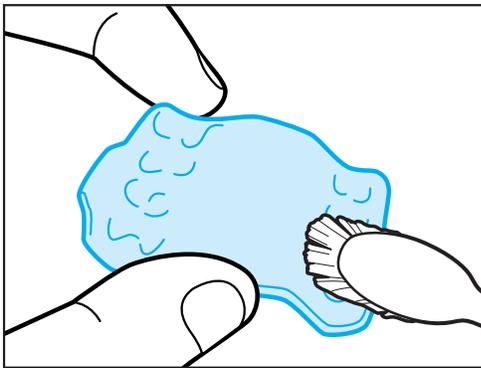

4. After the device removal:

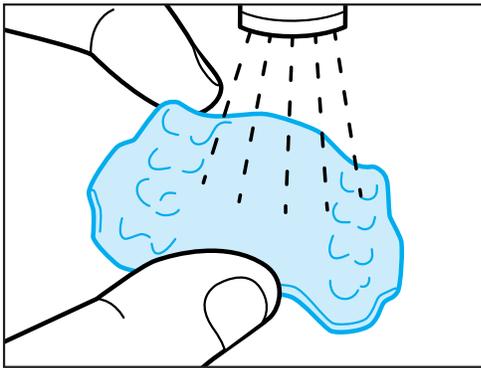


- Clean the device with a soft bristle toothbrush,

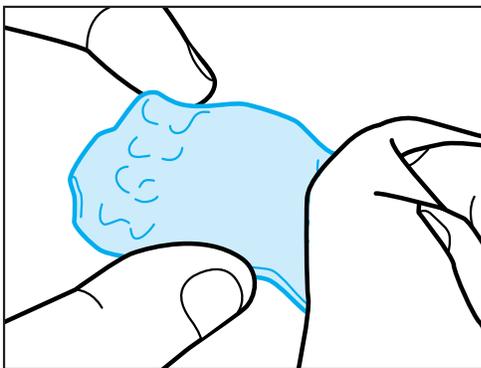

water and a small amount of toothpaste.

- Rinse the device thoroughly with running water.

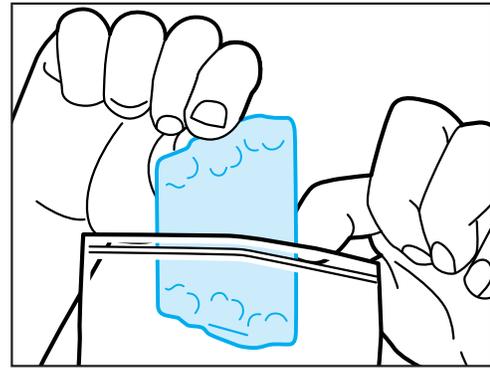

- Wipe the device with a tissue so that's its dry.

5. Do not throw away the used device. Store the device in its original bag.

Note:

- Immediately clean the device with toothpaste, toothbrush and water, thoroughly rinse and dry the device store it in its original bag.
- Take care in removing your expanders and holders.
- Do NOT use excessive force to bend or twist a device to get it off.
- Do NOT use any sharp object to remove your devices.
- Consult with your doctor if your devices are extremely difficult to remove.

CUSTOMER INFORMATION/USER INFORMATION

In case of a serious incident that directly or indirectly led, might have led or might lead to any of the following: a) the death of a patient, user or other person, b) the temporary or permanent serious deterioration of a patient's, user's or other person's state of health, c) a serious public health threat please immediately without any delay inform Align Technology and your health care professional.

DEVICE DISPOSAL

After your doctor confirms your treatment is complete, the Invisalign Palatal Expanders and Invisalign Palatal Holders should be disposed of in compliance with national and local laws.

FREQUENTLY ASKED QUESTIONS AND ANSWERS

Q: What if I lose or break one of my expanders or holders?

A: In the event that a device is lost or broken, you should immediately contact your doctor. Your doctor will probably tell you to start wearing your previous set or next set of devices immediately. He or she may order you a new set of devices (which should arrive in a few days) to replace the ones you just lost.

Q: Why did the doctor attach bumps to some of my teeth?

A: Bumps or "attachments" are bonded [i.e. glued] to help the device grip the teeth and aid with expansion. Devices have attachment wells or pockets that grip the bumps or "attachments" the doctor places on your teeth. The attachments are actually small pieces of composite or dental tooth colored material the doctor affixes to your teeth that are then gripped by attachment wells or pockets on your devices.